

Instructions

Section I: Sample Collection

- 1 Instruct the donor not to place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection.
- 2 Remove Sample Collector Protector.

- 3
 - Insert Sponge end of collector into mouth.
 - Actively swab the inside of the mouth and tongue with Sponge. As soon as the Sponge softens slightly, gently press the Sponge between the tongue and teeth to ensure **complete saturation**. The Sponge is saturated when no hard spots can be felt. Collect for at least three (3) minutes before removing Sponge.
 - Hand the saturated collector over to the tester.

At least 3 minutes

Section II: Testing

- 1
 - A. Align the Red Arrow on the device with either one of the **White Marks** on the Cap. Insert the collector **vertically** into the Cap and **press down firmly until the Cap reaches the thread**.
 - B. Twist the Handle clockwise 180° to tighten the Cap until the **Red Arrow** lines up with the other **White Mark**.
 - C. Place the test device horizontally on a clean and level surface with result window facing up.

1A. Correct

1B. Correct

1C. Correct

1A. Incorrect

1B. Incorrect

- 2 Read results at five (5) minutes. Results stable up to sixty (60) minutes.

10 minutes

Positive for COC
mAMP, PCP
OPI

Negative

Invalid

- 3 If results are positive, secure Cap with Security Seal and send to laboratory for confirmation.

Sample Port — Security Seal

Security Seal

* All positive results are presumptive and should be confirmed by an alternative method. The preferred confirmation method is GC/MS or GC/MS/MS.